



Kangaroo Joey Instruction

For questions contact 602-997-0500

Kangaroo Joey™ Set-Up Assembly

1. Attach the pump to the IV pole
2. Inform the patient to fill the feeding bag with formula prior to setting up the pump
3. Ensure that the top of the formula in the feeding bag is 6 inches above the top of the feeding pump prior to the patient beginning the feeding cycle



4. Set the feeding mode (EZ Mode or Normal Mode for the Flushing Mode Option):
 - a. Turn the pump on, then press the **Power** button in the lower right-hand corner



b. Once the kangaroo begins to jump across the screen, hold down the top left button



c . Push the **Biotech** option key



d. A "✓" will appear in front of the **EZ Pump Mode** if it's in this mode, otherwise the pump is in the **Normal Pump Mode**. Hit the corresponding key to check or uncheck



5. Select "Clear Settings"



6. Load the Feeding Set. Do not overstretch the tubing

a. Grasp the thumb tab on the valve and insert firmly into the left lower pocket



b. Press down on the thumb tab to ensure the valve is connected into place. The thumb tab should align with the slot in the blue door. The blue door will not close if the tab is not fully engaged



c. Grasp the black ring retainer

d. Gently wrap the tubing around the rotor

e. Insert the retainer directly into the upper pocket. **Do not overstretch the tubing**

f. Once the feeding set is loaded, close the blue door. The pump is now ready for operation



7. Priming the Pump
 - a. Press **“Prime Pump”**



- b. Press **“Auto Prime”** to automatically prime the pump set
 - c. Press the **“Hold to Prime Feed”** to manually top off the end of the feeding line if desired



- d. Press **“Done”**

8. Setting the Feed Rate

- a. Select **“Adjust Feed”** then **“Feed Rate”** and use the buttons on the left to program the pump from 1 to 400 ml. Select **“Enter”** when the desired rate is set

(i.e., *IsoSource 1.5 Cal @ 40 ml/hr x 24 hrs*)



- ** Note the Volume To Be Delivered **“VTBD”** Rate is an Optional Feature in the Normal Mode only. This option allows the pump to stop and alarm once a set amount of formula is delivered

(i.e., *Fibersource HN @ 60 ml/hr x 20 hrs or until 1200 ml is reached*)








Flush Set


9. Setting the Flush Rate in the Normal Mode Option Only
 - a. Select **“Adjust Flush”** then **“Flush Volume”** to set the volume of water per flush cycle from 10 to 500 ml. Select **“Enter”** when the desired rate is set

(i.e., *120 ml flush every 4 hrs*)

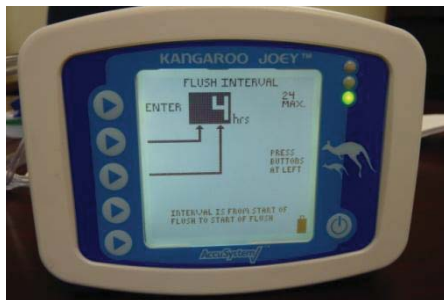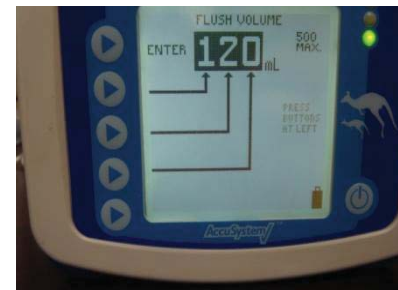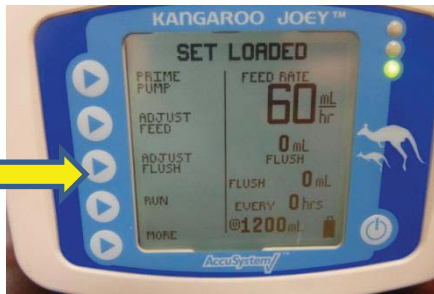

b. Select **“Flush Interval”** to set the interval time between the start of each flush cycle from 1 to 24 hours in increments of 1 hour. Select **“Enter”** then **“Done”**

(i.e., 30 ml auto flush every hr x 24 hrs)

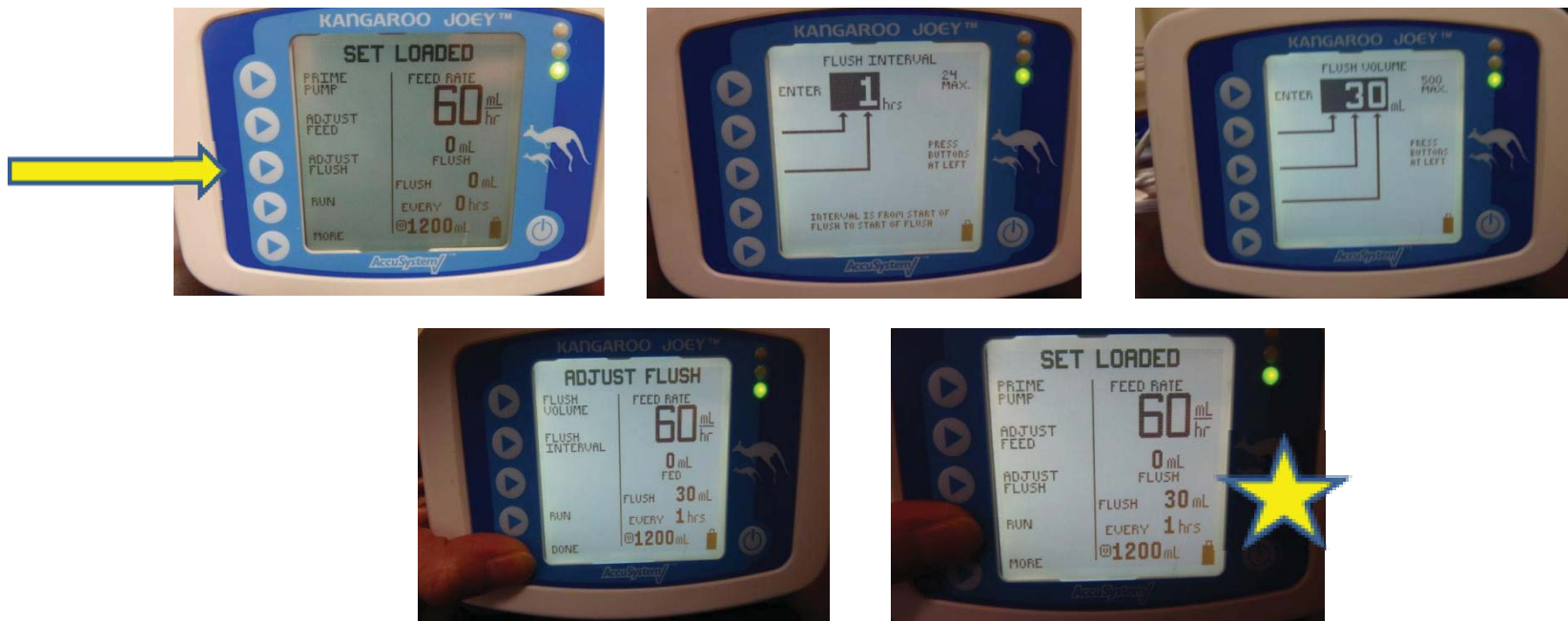

10. Select **“Run”** and a small drop will scroll down the screen indicating that the pump is running

